

02

Smart System Connect for Caregivers

Aloe Care



Help Is Here

Reach out to us for assistance with setup or answers to general questions; one of our team members will be happy to help. Call, email, or visit us online—


1-888-ALOECARE (1-888-256-3227)

support@aloecare.com

support.aloecare.com

Contents

1. Download	04
2. Log In	06
3. Get Connected	08
4. Invite Others	10

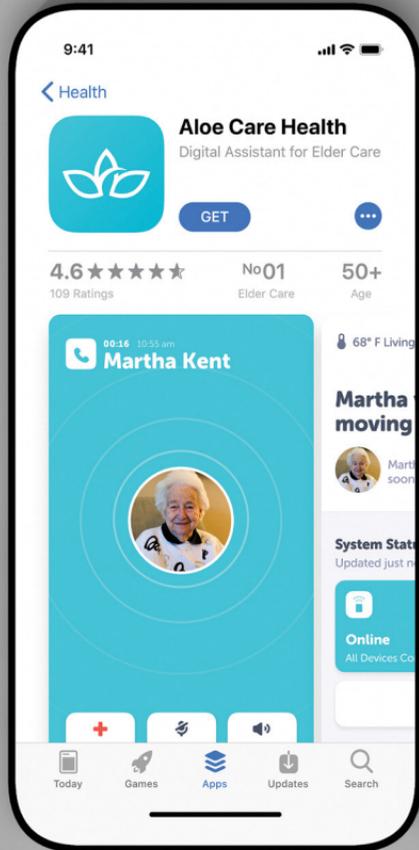



1

Download the App

Use the app to unlock all the Smart System benefits, including; wellbeing check-ins, medication management, coordination with other caregivers, and much more.

Download the Aloe Care Health app on a mobile device from the App Store or Google Play Store.



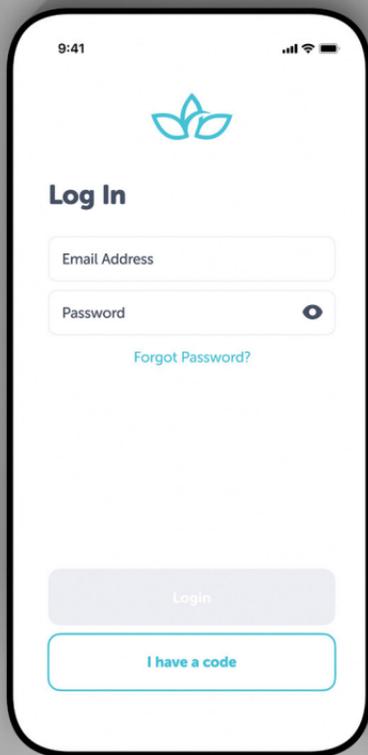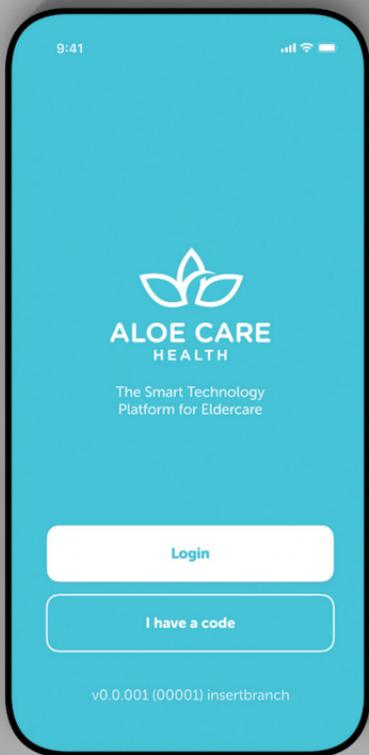

2


Log In to the App

Open the Aloe Care Health app, tap “login,” and enter your account credentials.

TIP Only a Primary Caregiver* (account holder) can perform initial set up within the app.

“Enter Invite Code” is for Secondary Caregivers (i.e., family members, friends, and professionals) to use when invited to the care circle.

*The Primary Caregiver is the person who ordered the Smart System and set up the account in their name.

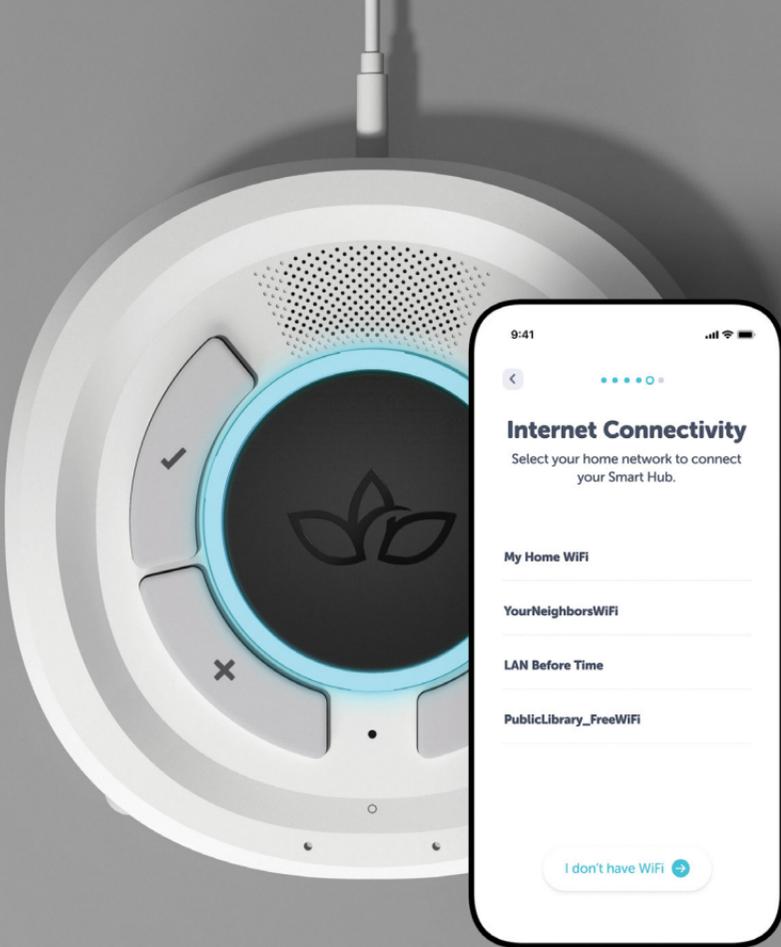

3



Get Connected

WiFi (recommended): Connect the Smart Hub to a home WiFi network to ensure the best call quality and coverage. Follow the in-app instructions to complete set up.

TIP Make sure you're near the Smart Hub when establishing an internet connection.

LTE: If there is no home WiFi network, tap "I don't have WiFi" to connect to the internet with LTE.

Once connected, the ring light will change from blue to white.

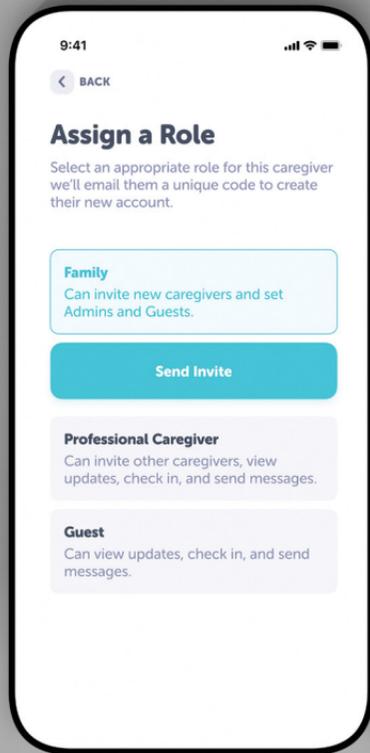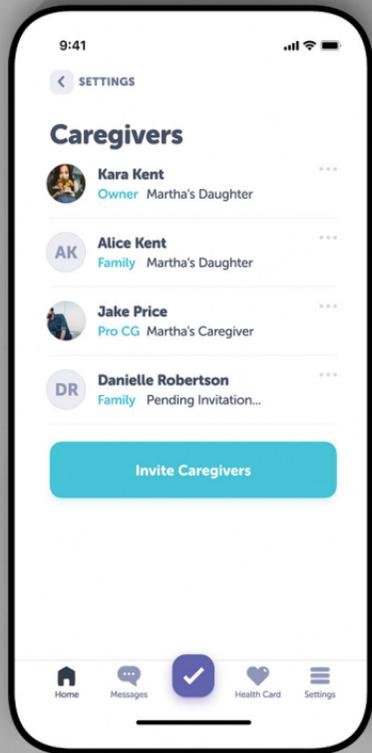

4

Expand Your Care Circle

Share the responsibility of caregiving with family, friends, and professionals. Go to the Caregiver menu in the app to add people and assign roles.



Setup Complete!

Read booklet **03, Get to Know**, to learn how to use your devices.

