

01

**Smart Fall Sensor  
Install & Setup**

Aloe Care



# Help Is Here

Reach out to us for assistance with setup or answers to general questions; one of our team members will be happy to help. Email, or speak to us directly—

[support@aloecare.com](mailto:support@aloecare.com)

1-888-ALOECARE (1-888-256-3227)

[support.aloecare.com](http://support.aloecare.com)

## Contents

1. Meet Your Fall Sensor	07
2. Find a Spot	08
3. Placement	09
4. Mount the Device	10
5. Get Connected	12
6. Teach the Device	14




## Safe and Comfortable at Home

Aloe Care is a streamlined communication ecosystem built to foster collaboration between caregivers and offer an improved stream of contact with independent adults. Our products utilize secure voice-communication and state-of-the-art privacy and data protection.

## The Smart Fall Sensor

Specifically designed to work in bathroom settings (the most common area for falls), the Smart Fall Sensor automatically detects falls by mapping the room using radiofrequency.

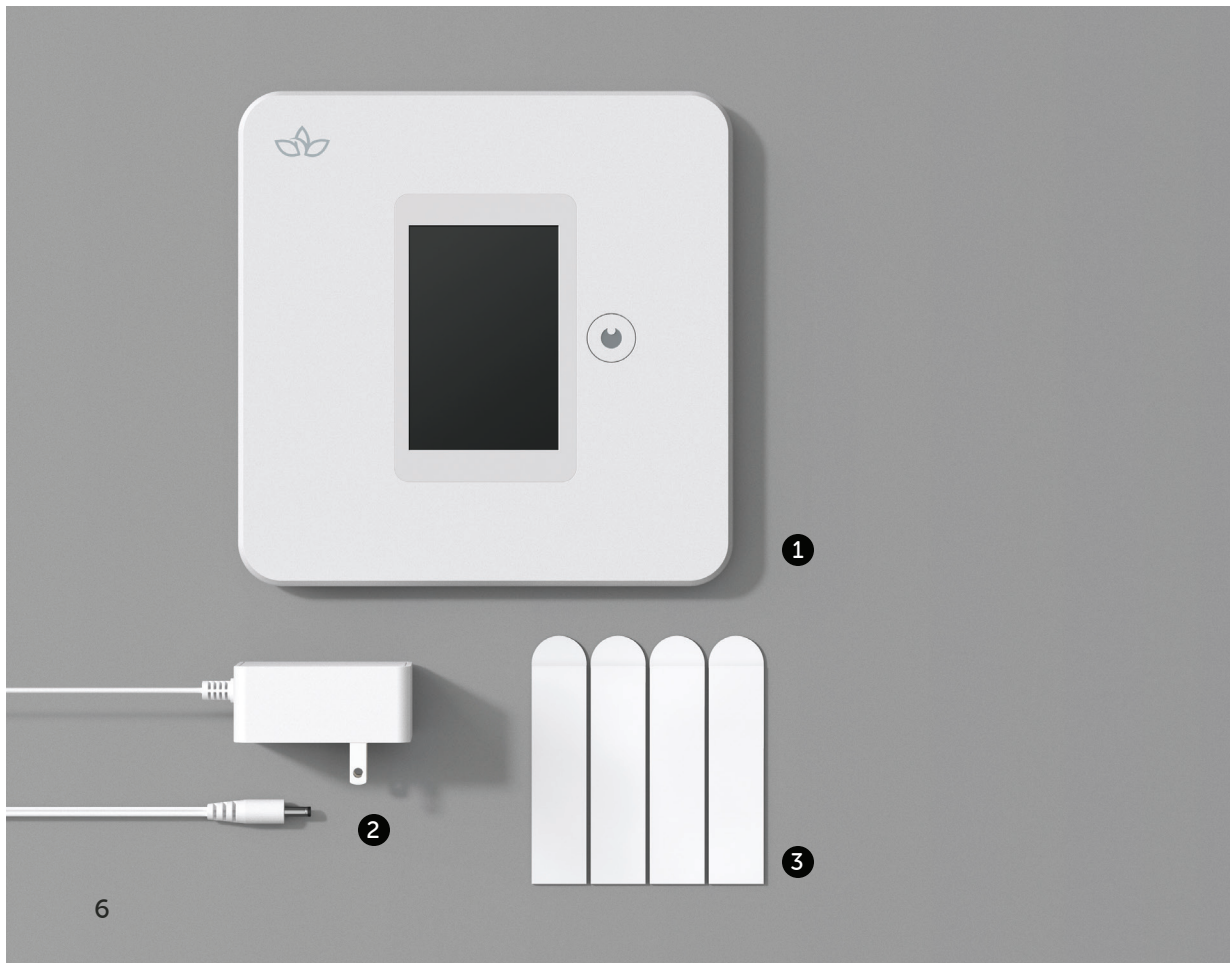

1

## Meet Your New Smart Fall Sensor

To begin, unpack all materials from the Fall Sensor box, including the device and accessories.

Make sure you have a measuring tape and a pencil, or a small piece of masking tape, to mark the wall.

Collect your items—

1. Fall Sensor
2. Power Cord
3. Double-sided Adhesives

2

## Find a Spot

Choose a central spot on a wall so the Fall Sensor has an unobstructed view of the entire bathroom.

For proper functionality—

- The device should be adhered flat against a wall, in the bathroom.
- The device should not be attached to the back of a door. It needs to be in a fixed location.
- The device should be placed 3 - 4 feet from an electrical outlet (power cord is 5 feet long).
- Contact with water should be avoided. The Fall Sensor is not waterproof.

8

3

## Placement

Your Fall Sensor operates most effectively when the center point of the device, measures 5 feet up from the bathroom floor.

After measuring the appropriate height, use a pencil or a piece of masking tape, to mark the correct height for your Fall Sensor.

**TIP** Before hanging the device, test out the length of your power cord. Make sure the cord reaches the desired position of the sensor and a nearby outlet.



4

## Mount the Device

Locate the adhesive placement guides on the back of your device. Apply the double-sided adhesive strips to the designated areas.

Next, clean the area of the wall where the device will be mounted. Remove the backing from all four strips, and mount the device in the location you've marked. Apply steady pressure against the device for 60 seconds.

**TIP** For proper orientation, make sure the center button is to the right of the touchscreen.





5

## Get Connected

Now that your Fall Sensor has been attached to a wall and plugged in, follow the on-screen instructions to connect your device to a WiFi network.

To complete device pairing—

1. A caregiver will need to access the Aloe Care Health mobile app.
2. Enter the unique code from your Fall Sensor's screen in the Aloe Care app to fully connect the Fall Sensor to your Care Space.



6

## Teach the Device

Now that your Fall Sensor is up and running, the device needs to learn the layout of your bathroom. Please follow the on-screen and voice instructions to successfully map your bathroom.

**TIP** The device will only map the room with one person in the room.



## Setup Complete!

Open booklet, **2. Get to Know** to learn how to use your device.



Aloe Care Health, Inc.  
Copyright © 2020 Aloe Care Health